Revision date: 2024-01, 030500003, Ver. A

Authorized Australian Sponsor

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Revision date: 2024-01, 030500003, Ver. A

Authorized Australian Sponsor

Compliance Management Solutions Pty Ltd
3/85 Curzon Street
North Melbourne Victoria 3051
Australia

Note: Line 2 in the frequency bands (E-GSM 900) is no longer valid!

Initializing of the blood pressure monitor

After the batteries are inserted, the blood pressure monitor switches itself on automatically and performs a display test.

Note

Make sure that all segments of the screen are displayed. Inform your healthcare provider/doctor if the display screen is damaged.

NOTICE

The device does not start!

If there is insufficient voltage, the device will not work.

For the (rechargeable) batteries, always ensure sufficient minimum voltage of 4.5 V to ensure that the blood pressure monitor works.

Tip

If the blood pressure monitor cannot complete the display test due to segments freezing, check the voltage of the inserted batteries.

The four batteries (AA, alkaline) must have a minimum voltage of **4.5 V**.

NOTICE

Leaking (rechargeable) batteries can cause damage to the device and property!

If the (rechargeable) batteries are stored for too long without the blood pressure monitor being used, this can cause the (rechargeable) batteries to leak.

Remove the (rechargeable) batteries from the battery compartment as soon as they have discharged and if the blood pressure monitor is **not** likely to be used for an extended period of time.

Authorized Australian Sponsor

Compliance Management Solutions Pty Ltd
3/85 Curzon Street
North Melbourne Victoria 3051
Australia

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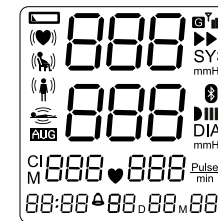
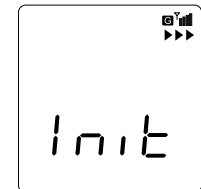
After the display test, the blood pressure monitor performs an initialization.

The display permanently shows **Init**.

Tip

During the initialization, the blood pressure monitor searches for an available network and records the current time. The initialization **cannot** be interrupted and ends automatically.

If there is high network availability, the initialization will complete within a few seconds.

If it is not possible to establish a mobile network connection, the initialization will end after a maximum of 45 minutes (timeout). The error message **Err Init** is permanently shown on the display, see chapter *Display information*. It is not possible to monitor blood pressure in this status.

If the initialization is successful, the start screen is displayed.

Display information

Display	(Possible) Cause	Measure
	Registration in mobile network not possible. Destination address not available. It is not possible to set the current time.	Move to a location with better mobile network signal, remove the batteries, wait a few minutes and reinsert the batteries to start initialization. If the message still appears after following the instructions, contact your healthcare provider/doctor.
	Initialization is performed, see chapter <i>Initializing of the blood pressure monitor</i> .	During the initialization, the blood pressure monitor searches for an available network and records the current time. The initialization cannot be interrupted and ends automatically. If there is high network availability, the initialization will complete within a few seconds. If it is not possible to establish a mobile network connection, the initialization will end after a maximum of 45 minutes (timeout). The error message Err Init is permanently shown on the display. It is not possible to monitor blood pressure in this status.

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